



DE	Blutdruckmessgerät Gebrauchsanweisung	2
EN	Blood pressure monitor Instructions for use	20
FR	Tensiomètre Mode d'emploi	38
ES	Tensiómetro Instrucciones de uso	57
IT	Misuratore di pressione Istruzioni per l'uso	75
TR	Tansiyon ölçme cihazı Kullanım kılavuzu	93
RU	Прибор для измерения кровяного давления Инструкция по применению	111
PL	Ciśnieniomierz Instrukcja obsługi	130



Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards, your Beurer team

Table of contents

1. Included in delivery.....	20
2. Signs and symbols	21
3. Intended use.....	22
4. Warnings and safety notes	22
5. Device description.....	24
6. Initial use.....	25
7. Usage	27
8. Cleaning and maintenance.....	33
9. Accessories and replacement parts	33
10. What if there are problems?	34
11. Disposal	34
12. Technical specifications	35
13. Warranty/service.....	36









1. Included in delivery





Check that the exterior of the delivery cardboard packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 x Blood pressure monitor
- 1 x Upper arm cuff
- 4 x 1.5V LR03 AAA batteries
- 1 x Storage bag
- 1 x Instructions for use
- 1 x Quick Start Guide

2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	WARNING Warning notice indicating a risk of injury or damage to health
	IMPORTANT Safety note indicating possible damage to the device/accessory
	Note Note on important information
	Observe the instructions for use
	Application part, type BF
	Direct current
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Do not dispose of batteries containing hazardous substances with household waste

	Dispose of packaging in an environmentally friendly manner
	Manufacturer
Storage/Transport 	Permissible storage and transport temperature and humidity
Operating 	Permissible operating temperature and humidity
IP21	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water
SN	Serial number
CE 0483	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical devices.

3. Intended use

The BM 54 upper arm blood pressure monitor is intended for home use and serves as a non-invasive way of measuring and monitoring arterial blood pressure and pulse values in adults with an upper arm circumference of 22-44 cm.

As a result, you can quickly and easily measure your blood pressure and pulse and view various average values from previous measurements. The recorded measurements are classified and evaluated graphically. A symbol in the display also warns of possible existing cardiac arrhythmia.

4. Warnings and safety notes



Notes on use

- In order to ensure comparable values, always measure your blood pressure at the same time of day.
- Do not take a measurement within 30 minutes of eating, drinking, smoking or exercising.
- Before the initial blood pressure measurement, make sure always to rest for about 5 minutes.
- Furthermore, if you want to take several measurements in succession, make sure always to wait for at least 1 minute between the individual measurements.
- Repeat the measurement if you are unsure of the measured value.
- The measurements taken by you are for your information only – they are no substitute for a medical examination! Discuss the measured values with your doctor and never base any medical decisions on them (e.g. medicines and their dosages).

- Using the blood pressure monitor outside your home environment or whilst on the move (e.g. whilst travelling in a car, ambulance or helicopter, or whilst undertaking physical activity such as playing sport) can influence the measurement accuracy and cause incorrect measurements.
- Do not use the blood pressure monitor on newborns or patients with preeclampsia. We recommend consulting a doctor before using the blood pressure monitor during pregnancy.
- Cardiovascular diseases may lead to incorrect measurements or have a detrimental effect on measurement accuracy. The same also applies to very low blood pressure, diabetes, circulatory disorders and arrhythmias as well as chills or shaking.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who is responsible for their safety or are instructed by such a person in how to use the device. Supervise children around the device to ensure they do not play with it.
- The blood pressure monitor must not be used in connection with a high-frequency surgical unit.
- Only use the device on people that have the upper arm circumference specified for the device.
- Please note that when inflating, the functions of the limb in question may be impaired.
- During the blood pressure measurement, the blood circulation must not be stopped for an unnecessarily long time. If the device malfunctions remove the cuff from the arm.
- Avoid any mechanical restriction, compression or bending of the cuff line.



- Do not allow sustained pressure in the cuff or frequent measurements. The resulting restriction of the blood flow may cause injury.
- Make sure that the cuff is not placed on an arm in which the arteries or veins are undergoing medical treatment, e.g. intravascular access or intravascular therapy, or an arteriovenous (AV) shunt.
- Do not use the cuff on people who have undergone a mastectomy.
- Do not place the cuff over wounds as this may cause further injury.
- Place the cuff on your upper arm only. Do not place the cuff on other parts of the body.
- Please note that data can only be transferred and stored when your blood pressure monitor is supplied with power. As soon as the batteries are empty, the blood pressure monitor loses the date and time.
- To conserve the batteries, the blood pressure monitor switches off automatically if you do not press any buttons for 30 seconds.
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or incorrect use.

Notes on storage

- The blood pressure monitor is made from precision and electronic components. The accuracy of the measured values and service life of the device depend on its careful handling:
 - Protect the device from impacts, moisture, dirt, marked temperature fluctuations and direct sunlight.

- Do not drop the device.
- Do not use the device in the vicinity of strong electromagnetic fields and keep it away from radio systems or mobile telephones.
- Only use the cuff included with the delivery or original replacement cuffs. Otherwise incorrect measured values will be recorded.


Notes on handling batteries


- If your skin or eyes come into contact with battery fluid, rinse the affected area with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Therefore, store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
-  **Risk of explosion!** Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

Notes on electromagnetic compatibility


- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

5. Device description

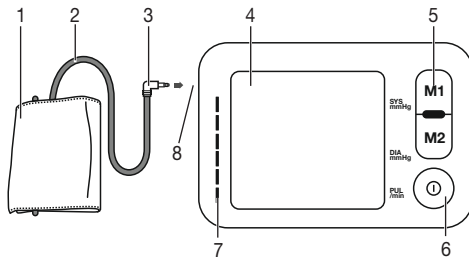

1. Cuff
2. Cuff line
3. Cuff connector
4. Display
5. Memory buttons **M1/M2**
6. START/STOP button 
7. Risk indicator
8. Connection for cuff connector (left-hand side)

Information on the display:

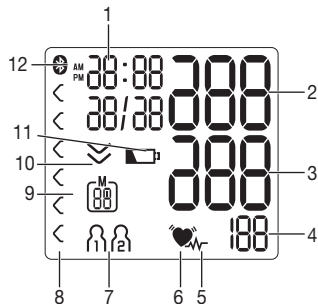

1. Time/date
2. Systolic pressure
3. Diastolic pressure
4. Calculated pulse value
5. Cardiac arrhythmia symbol
6. Pulse symbol
7. User memory
8. Risk indicator
9. Number of memory space/memory display for average value (M), morning (M^M), evening (M^E)
10. Release air (arrow)
11. Battery display symbol
12. Symbol for *Bluetooth*® transfer

System requirements for the “beurer HealthManager” app

- iOS ≥ 10.0 / Android™ ≥ 5.0
- *Bluetooth*® ≥ 4.0


List of compatible devices:



6. Initial use

6.1 Inserting the batteries

- Remove the battery compartment lid on the rear of the device.
- Insert four 1.5 V AAA (alkaline type LR03) batteries. Make sure that the batteries are inserted the correct way round in accordance with the markings. Do not use rechargeable batteries.
- Close the battery compartment lid again carefully.
- All display elements are briefly displayed, 24 h or 12 h flashes in the display. Now set the date and time as described below. If the battery replacement symbol is permanently displayed, you can no longer perform any measurements and must replace all batteries. Once the batteries have been removed from the device, the date and time must be set again. Any saved measured values are retained.

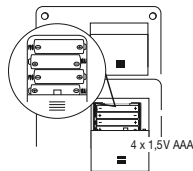

6.2 Setting the hour format, date, time and Bluetooth®

It is essential that you set the date and time. Otherwise, you will not be able to save your measured values correctly with a date and time and access them again later.

- i** There are two different ways to access the menu from which you can adjust the settings:
- Before initial use and after each time you replace the battery:
When inserting batteries into the device, you will be taken to the relevant menu automatically.
- If the batteries have already been inserted:
With the device **switched off**, press and hold the **START/STOP** button **ⓘ** for approx. 5 seconds.

In this menu you can adjust the following settings in succession:

Hour format → **Date** → **Time** → **Bluetooth®**

Hour format

The hour format flashes on the display.

- Select the desired hour format using the **M1/M2** memory buttons and confirm with the **START/STOP** button **ⓘ**.

- i** If you press and hold the **M1** or **M2** memory button, you can set the values more quickly.



Date

The year flashes on the display.

- Select the year using the **M1/M2** memory buttons and confirm with the **START/STOP** button **ⓘ**.



The month flashes on the display.

- Select the month using the **M1/M2** memory buttons and confirm with the **START/STOP** button **ⓘ**.



The day flashes on the display.

- Select the current day using the **M1/M2** memory buttons and confirm with the **START/STOP** button **ⓘ**.



- i** If you have set the 12h hour format, the month is displayed before the day.

Time

The hours flash on the display.

- Select the current hours using the **M1/M2** memory buttons and confirm with the **START/STOP** button **ⓘ**.



The minutes flash on the display.

- Select the current minutes using the **M1/M2** memory buttons and confirm with the **START/STOP** button **ⓘ**.



The *Bluetooth*® symbol flashes on the display.

- Use the **M1/M2** memory buttons to select whether automatic *Bluetooth*® data transfer is to be activated (*Bluetooth*® symbol flashes) or deactivated (*Bluetooth*® symbol is not shown) and confirm with the **START/STOP** button ①.


① *Bluetooth*® transfers will reduce the battery life.

7. Usage


Ensure the device is at room temperature before measuring. The measurement can be performed on the left or right arm.

7.1 Attaching the cuff


Place the cuff onto the bare upper arm. The circulation of the arm must not be hindered by tight clothing or similar.

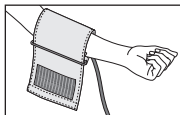


The cuff must be placed on the upper arm so that the bottom edge is positioned 2 – 3 cm above the elbow and above the artery. The line should point to the centre of the palm.

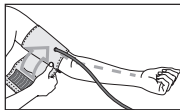


Now tighten the free end of the cuff, but make sure that it is not too tight around the arm and close the hook-and-loop fastener. The cuff should be fastened so that two fingers fit under the cuff.

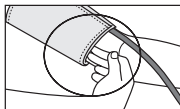

Now insert the cuff line into the connection for the cuff connector.

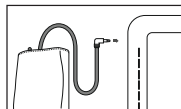

The cuff is suitable for you if the index mark (▼) is within the OK range after fitting the cuff.

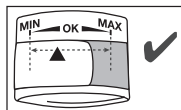


- ① If the measurement is performed on the right upper arm, the line should be located on the inside of your elbow. Ensure that your arm is not pressing on the line.

Blood pressure may vary between the right and left arm, which may mean that the measured blood pressure values are different. Always perform the measurement on the same arm. If the values between the two arms are significantly different, please consult your doctor to determine which arm should be used for the measurement.

Important: The device may only be operated with the original cuff. The cuff is suitable for an arm circumference of 22 to 44 cm.

7.2 Adopting the correct posture

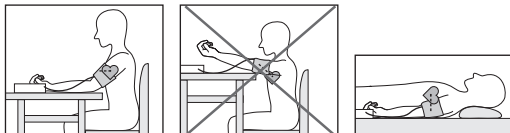

- Before the initial blood pressure measurement, make sure always to rest for about 5 minutes. Otherwise deviations can occur.
- You can take the measurement while sitting or lying. Always make sure that the cuff is at heart level.
- To take your blood pressure, make sure you are sitting comfortably with your arms and back leaning on something. Do not cross your legs. Place your feet flat on the ground.
- To avoid falsifying the measurement, it is important to remain still during the measurement and not to speak.
- Wait for at least 1 minute before taking another measurement.

7.3 Taking a blood pressure measurement


As described above, attach the cuff and adopt the posture in which you want to perform the measurement.

- Press the **START/STOP** button  to start the blood pressure monitor. All display elements are briefly displayed.




The blood pressure monitor will begin the measurement automatically after 3 seconds.

- You can cancel the measurement at any time by pressing the **START/STOP** button .



As soon as a pulse is found, the pulse symbol  will be displayed.

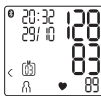
- Systolic pressure, diastolic pressure and pulse measurements are displayed.




- **Er** appears if the measurement could not be performed properly (see chapter “10. What if there are problems?”). Repeat the measurement.

Er

- Now select the desired user memory by pressing the **M1** or **M2** memory buttons. If you do not select a user memory, the measurement is stored in the most recently used user memory. The relevant  or  symbol appears on the display.

- Press the **START/STOP** button  to switch off the blood pressure monitor. The measurement is then stored in the selected user memory.
- If the device is not switched off manually, it will switch off automatically after 30 seconds.

If **Bluetooth® data transfer** has been activated, then after having confirmed the user memory by pressing the **START/STOP** button , the data is transferred to the “beurer HealthManager” app (see chapter “7.6 Transferring measurements”).

- ① When connecting for the first time, a randomly generated six-digit PIN code is displayed on the device, and at the same time an input field appears on the smartphone in which you must enter this six-digit PIN code. After successfully entering the code, the device will be connected to your smartphone.
- The **Bluetooth®** symbol on the display flashes and the blue LED lights up. The blood pressure monitor now attempts to connect to the app for approx. 30 seconds.
- The **Bluetooth®** symbol stops flashing as soon as a connection is established. All measurement data is automatically transferred to the app. Once the data has been successfully transferred, the device switches off. If the data transfer was unsuccessful, the blue LED goes out and “Er” appears on the display.

- If a connection to the app cannot be established after 30 seconds, the **Bluetooth®** symbol goes out and the blood pressure monitor switches off automatically after 3 minutes.
- ① Please note that you must add the blood pressure monitor to “My devices” in the “beurer HealthManager” app to enable data transfer. The “beurer HealthManager” app must be activated to allow data transfer. If the latest data is not displayed on your smartphone, repeat the data transfer as described in chapter 7.6.


If you forget to turn off the blood pressure monitor, it will switch off automatically after approx. 30 seconds. In this case too, the value is stored in the selected or most recent user memory and the data is transferred if **Bluetooth®** data transfer has been activated.

- Wait for at least 1 minute before taking another measurement.





7.4 Evaluating the results

Cardiac arrhythmia:

This device can identify potential disruptions of the heart rhythm when measuring and, if necessary, indicates this after the measurement with the symbol .

This can be an indicator for arrhythmia. Arrhythmia is a condition in which the heart rhythm is abnormal because of flaws in the bioelectrical system that regulates the heartbeat. The symptoms (skipped or premature heart beats, pulse being slow or too fast)

can be caused by factors such as heart disease, age, physical make-up, excess stimulants, stress or lack of sleep. Arrhythmia can only be determined through an examination by your doctor. If the symbol  is shown on the display after the measurement has been taken, repeat the measurement. Please ensure that you rest for 5 minutes beforehand and do not speak or move during the measurement. If the symbol  appears frequently, please consult your doctor.

Self-diagnosis and treatment based on the measurements can be dangerous. Always follow your doctor's instructions.

Risk indicator:

The measurements can be classified and evaluated in accordance with the following table.

However, these standard values serve only as a general guideline, as the individual blood pressure varies in different people and different age groups etc.

It is important to consult your doctor regularly for advice. Your doctor will tell you your individual values for normal blood pressure as well as the value above which your blood pressure is classified as dangerous.

The classification on the display and the scale on the device show which category the recorded blood pressure values fall into. If the values of systole and diastole fall into two different categories (e.g. systole in the 'High normal' category and diastole in the 'Normal' category), the graphical classification on the device always shows the higher category; for the example given this would be 'High normal'.

Blood pressure value category		Systole (in mmHg)	Diastole (in mmHg)	Action
Level 3: severe hypertension	red	≥ 180	≥ 110	Seek medical attention
Level 2: moderate hypertension	orange	160–179	100–109	Seek medical attention
Level 1: mild hypertension	yellow	140–159	90–99	Regular monitoring by doctor
High normal	green	130–139	85–89	Regular monitoring by doctor
Normal	green	120–129	80–84	Self-monitoring
Optimal	green	< 120	< 80	Self-monitoring

Source: WHO, 1999 (World Health Organization)

7.5 Displaying and deleting measured values

User memory

The results of every successful measurement are stored together with the date and time. The oldest measurement is overwritten in the event of more than 60 measurements.



- Using the memory button **M1** or **M2**, select the desired user memory (M1 M2) when the device is switched off.
- To view the measurements for user memory M1, press the **M1** memory button.
- To view the measurements for user memory M2, press the **M2** memory button.

A flashes on the display.

The average value of all saved measured values in this user memory is displayed.



Average values

- ❗ If *Bluetooth*® is activated (the  symbol flashes on the display), the blood pressure monitor attempts to connect to the app. If you press the **M1** button in the meantime, the transfer is cancelled and the average values are displayed. If you press the **M2** button, the transfer is cancelled and the measurement data of user memory M2 is displayed. The  symbol is no longer shown. The buttons are deactivated as soon as a connection is established and the data is transferred.

Average values

- Press the relevant memory button (**M1** or **M2**)

AM flashes on the display.

The average value of the morning measurements for the last 7 days is displayed (morning: 5 a.m. – 9 a.m.).





- Press the relevant memory button (**M1** or **M2**).

PM flashes on the display.

The average value of the evening measurements for the last 7 days is displayed (evening: 6 p.m. – 8 p.m.).



Individual measured values

- If you press the relevant memory button (**M1** or **M2**) again, the last individual measurement is displayed (in this example, measurement 03).
- If you press the relevant memory button (**M1** or **M2**) once more, you can view all previous individual measurements.
- To switch the device off again, press the START/STOP button .
- ❗ You can exit the menu at any time by pressing the **START/STOP** button .



- To clear the relevant user memory, you must first select a user memory.
- Start the retrieval of the average measured values. R flashes on the display and the average value of all saved measured values in this user memory is displayed.
- Press and hold the memory button **M1** or **M2** for 5 seconds depending on the user memory you are in.

All the values in the current user memory are deleted.



- To clear individual measurements from the relevant user memory, you must first select a user memory.
- Start the retrieval of the individual measured values.
- Press and hold the memory button **M1** or **M2** for 5 seconds (depending on the user memory you are in).
- The selected value is deleted. The device briefly displays CL 00.
- If you would like to delete other values, repeat the process described above.

You can switch off the device at any time by pressing the **START/STOP** button.

7.6 Transferring measurements

Transfer via *Bluetooth*® low energy technology

It is also possible to transfer the measured values saved on the device to your smartphone using *Bluetooth*® low energy technology.

You will need the “beurer HealthManager” app for this. The app is available free of charge on the Apple App Store and from Google Play.

Proceed as follows to transfer values:

If *Bluetooth*® is activated in the settings menu, the data is transferred automatically after the measurement. The ⓘ symbol appears in the top left of the display (see chapter “7.3 Taking a blood pressure measurement”).

- ⓘ When connecting for the first time, a randomly generated six-digit PIN code is displayed on the device, and at the same time an input field appears on the smartphone in which you must enter this six-digit PIN code. After successfully entering the code, the device will be connected to your smartphone



Step 1: BM 54

Activate *Bluetooth*® on your device (see chapter “6.2 Setting the hour format, date, time and *Bluetooth*®”).



Step 2: “beurer HealthManager” app

In the “beurer HealthManager” app, add the BM 54 under “Settings/My devices”.




Step 3: BM 54

Take a measurement.



Step 4: BM 54


Data transfer immediately following a measurement:

- If ***Bluetooth*® data transfer** has been activated, the data is transferred after having confirmed the user memory by pressing the **START/STOP** button .

Step 4: BM 54

Data transfer at a later point:

- Go to memory mode (chapter 7.5). Select the desired user memory. The *Bluetooth*® transfer starts automatically.

-  The “beurer HealthManager” app must be activated to allow data transfer.

If your smartphone has a protective cover, remove this to ensure that there is no interference during the transfer. Begin the data transfer in the “beurer HealthManager” app.

8. Cleaning and maintenance

- Clean the device and cuff carefully using a slightly damp cloth only.
- Do not use any cleaning agents or solvents.
- Under no circumstances hold the device and cuff under water, as this can cause liquid to enter and damage the device and cuff.
- If you store the device and cuff, do not place heavy objects on the device and cuff. Remove the batteries. The cuff line should not be bent sharply.


9. Accessories and replacement parts

Accessories and replacement parts are available from the corresponding service address (according to the service address list). Please state the corresponding order number.

Designation	Item number and/or order number
Universal cuff (22-44 cm)	163.952

10. What if there are problems?

Error message	Possible cause	Solution
Er1	Unable to record a pulse.	Please wait one minute and repeat the measurement. Ensure that you do not speak or move during the measurement.
Er2	You moved or spoke during the measurement.	
Er3	The cuff was not attached correctly.	Please observe the information in chapter "7.1 Attaching the cuff".
Er4	An error occurred during the measurement.	Please wait one minute and repeat the measurement. Ensure that you do not speak or move during the measurement.


Error message	Possible cause	Solution
Er5	The inflation pressure is higher than 300 mmHg.	Please take another measurement to check whether the cuff can be correctly inflated. Make sure that neither your arm nor other heavy objects are pressing on the line, and that the line is not bent.
Er6 	The batteries are almost empty.	Insert new batteries into the device.
Er7	Unable to transfer the data via <i>Bluetooth</i> ®.	Please observe the information in chapter "7.6 Transferring measurements".

11. Disposal

Disposing of the batteries

- Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

- The codes below are printed on batteries containing harmful substances:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury

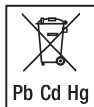

i Repairing and disposing of the device

- Do not repair or adjust the device yourself. Proper operation can no longer be guaranteed in this case.
- Repairs must only be carried out by Customer Services or authorised retailers.
- Do not open the device. Failure to comply will invalidate the warranty.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.



12. Technical specifications

Model no.	BM 54
Measurement method	Oscillometric, non-invasive blood pressure measurement on the upper arm
Measurement range	Cuff pressure 0-250 mmHg, systolic 50-250 mmHg, diastolic 30-200 mmHg, pulse 40-180 beats/minute

Display accuracy	Systolic ± 3 mmHg, diastolic ± 3 mmHg, pulse $\pm 5\%$ of the value shown
Measurement uncertainty	Max. permissible standard deviation according to clinical testing: systolic 8 mmHg / diastolic 8 mmHg
Memory	2 x 60 memory spaces
Dimensions	L 139 mm x W 94 mm x H 48 mm
Weight	360 g (without batteries, with cuff)
Cuff size	22 to 44 cm
Permissible operating conditions	+5°C to +40°C, 15-93% relative humidity (non-condensing)
Permissible storage conditions	-25°C to +70°C, $\leq 93\%$ relative humidity, 700-1060 hPa ambient pressure
Power supply	4x 1.5 V AAA batteries
Battery life	Lasts for approx. 200 measurements depending on the blood pressure and inflation pressure as well as the number of <i>Bluetooth</i> ® connections.
Classification	Internal supply, IP21, no AP or APG, continuous operation, application part type BF

Data transfer via Bluetooth® wireless technology	The blood pressure monitor uses Bluetooth® low energy technology, 2402 MHz – 2480 MHz frequency band, max. 4 dBm transmission power, compatible with Bluetooth® 4.0 smart-phones/tablets
--	--

The serial number is located on the device or in the battery compartment.

Technical information is subject to change without notification to allow for updates.

- This device complies with the European standard EN60601-1-2 (in compliance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-8) and is subject to special precautionary measures with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this device.
- The device complies with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktegesetz) and the standards EN1060-1 (Non-invasive sphygmomanometers – Part 1: General requirements), EN1060-3 (Non-invasive sphygmomanometers – Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems) and IEC 80601-2-30 (Medical electrical equipment – Part 2 – 30: Particular requirements for the basic safety and essential performance of automated non-invasive sphygmomanometers).
- The accuracy of this blood pressure monitor has been carefully checked and developed with regard to a long useful life. If the device is used for commercial medical purposes, it must be

regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

- We hereby confirm that this product complies with the European RED Directive 2014/53/EU. The CE Declaration of Conformity for this product can be found under: www.beurer.com/web/we-landingpages/de/cedeclarationof-conformity.php.

13. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 5 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instruc-

tions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;

- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.